

THANK YOU!

On behalf of all of us at IRadimed®, thank you for purchasing the MRidium® MRI IV Infusion System...iRadimed's solution for ensuring the **'Continuity of Care'** for patients requiring continuous or intermittent infusions throughout their MRI care cycle. Our Applications Program has been specifically designed to promote an effective and safe product implementation of the MRidium® IV Infusion System into your clinical practice.



CLINICAL APPLICATIONS PROGRAM

Inside this Brochure:

Classroom Methodology	2
General Classes	2
'Super User' Class	3
Training Outline	3
On-Line Training	4
Training Modules	4
Disposable IV Set Options	4

THE TRAINING YOU NEED FOR CLINICAL SAFETY AND SUCCESS

To promote safe and effective product implementations, IRadimed® delivers a comprehensive clinical applications program. We partner with you to provide the support and resources you need to help ensure your clinicians are effectively trained on the MRidium® MRI IV Infusion System .

'SUPER USER' CLASSROOM



WHY 'SUPER USERS' ?

'Super Users' will serve as your first level of support, providing guidance and continuing education on the highly specialized MRidium® MRI IV Infusion System.

OBJECTIVE: Continuing Education Resource

We will prepare your appointed 'Super Users' to know every aspect of the MRidium® MRI IV Infusion System. Additionally, we will provide the task knowledge and extensive 'Hands-On' training and support necessary for your 'Super Users' to facilitate future training for new hires and continuing education for your current staff.

WHO?

When appointing your 'Super Users' look for these qualities:

1. Effective Communicator
2. Critical Thinker
3. Problem Solver
4. Detailed Oriented
5. Love of Learning

'Super Sessions' with 4 to 6 'Super Users' per class, may last from up to 45 minutes to one hour.

TRAINING SUPPORT TOOLS

To assist your trainers, a training toolkit will be provided. The toolkit consists of helpful items to assist in making the training classes successful including, training materials, check-lists and hands-on practice materials. These resources will present a professional approach to the learning experience.

MRidium® MRI IV INFUSION SYSTEM TRAINING OUTLINE

1. Introducing the MRidium® MRI IV Infusion System
 - a. System Introduction: Pump, Sidecar, Remote, SpO₂ etc.
 - b. Infusions within the MRI Suite - equipment placement, safety and gauss limits.
2. Selecting the Appropriate IV Set
3. Priming the IV Set
 - a. Flow Preventer Clamp
 - b. Care and Handling of the IV Sets during Priming
4. Loading the 'Pump'/ IV Set Installation
5. Programming
 - a. Rate and Volume
 - b. Dose Rate Calculator
 - c. DERS Drug Library (if applicable)
6. Using your MRidium® Remote Control (if applicable)
7. SpO₂ (if applicable)
8. Alarm and Alert Resolution



MRidium® 3865 Remote

Classroom Focused Training

Our focused training methodology is customized to meet the needs of each facility. We have found after many years of experience that conducting training sessions on the physical care unit has proven to be very disruptive. Clinicians are constantly distracted by alarms, call lights and telephones.....making it difficult for them to focus on learning new technology. Alternatively, conducting sessions in a classroom setting provides an optimal learning environment, one which **minimizes disruptions** and **maximizes attention**. IRadimed® offers two types of classroom formats: '**General**' and '**Super User**'

"The MRidium® MRI Infusion System is very easy to use... requiring minimal set-up time."

GENERAL CLASSROOM WHO SHOULD ATTEND?

The MRidium® IV Infusion System is highly specialized and only intended for use within the MRI department. Keep this in mind when determining who will attend training. While we realize many patient care units may transport a patient to the MRI on the MRidium®.....it is recommended that only those who will have regular contact with the pump attend the training sessions. For those who will have infrequent contact with the system, it is suggested they take advantage of our 'On-line' Training Modules or attend a class conducted by the facilities 'Super User' on a future date.

OBJECTIVES?

At the end of the session participants will have the clinical knowledge to correctly set-up and administer IV infusions utilizing the MRidium® MRI IV System within the MRI Suite. Class attendee's will receive Clinical Applications training as outlined within this brochure along with '**step by step**' demonstrations of:

1. **Selecting, handling and correctly priming the IV Set**
2. **Loading the Pump**
3. **Programming the Infusion**
4. **Setting up the Remote Control (if applicable)**


HOW LONG WILL IT TAKE?

The number of sessions is dependent on the departments and staff to be trained. 'General Sessions' are typically larger groups of clinicians from those patient care units which will interface on a regular basis with the MRidium IV pump. Classroom times generally run up to 30 minutes depending on the size of the class.

MRidium® 3860+ IV Pump



We're on the web!
www.iradimed.com

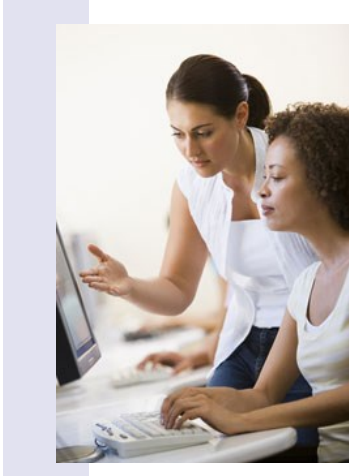

On-Line Training Modules

IRadimed® offers Comprehensive **'On-Line'** MRidium® IV Infusion System Training Modules... allowing students to view educational material from any computer within your facility or from home. These training modules provide a supplemental teaching aid for clinical trainers as well as continuing educational use.

As a nurse, clinician, or MRI Tech, you can use these modules to learn something new about your MRidium® IV Infusion System or simply use as a refresher course **at your convenience**.

Train your staff! Become an expert!

Feel confident in your ability to provide 'Continuity of Care' for critically ill patient's undergoing MRI procedures.

The MRidium® Video Modules:

1. Selecting the Appropriate IV Infusion Set
2. Loading the Infusion Pump
3. Programming the Infusion
4. Alarms, Alerts and Prompts
5. Optional SpO₂ Monitoring

HAVE YOU ORDERED YOUR IV SETS?

Be sure to place your order for IV sets prior to your training dates so as to be prepared for immediate implementation of your new MRidium® MRI IV Infusion system. Should you have any

Disposable IV Infusion Sets TO ORDER CALL 866.677.8022

Spike Infusion Set, Part Number 1056
 Latex-Free INFUSION Set WITH SPIKE AND TWO NEEDLE-FREE INJECTION PORTS

Vented Syringe Set, Part Number 1057
 Latex-Free VENTED SYRINGE ADAPTER SET, Microbore 4mL Prime Volume

NOTE: SYRINGE NOT INCLUDED

Extension Set, Part Number 1058
 Latex-Free EXTENSION SET WITH ONE NEEDLE-FREE INJECTION PORT

questions in regard to which IV Sets to order please contact your Iradimed® Sales Manager for guidance.

